

FirstView[®]

Bioline
DIAGNOSTICS
Your safety. Our commitment.


Rapid test for HIV-Ab, Ag 4th Gen

For Qualitative Detection of HIV-Ab/Ag 4th Gen Human Serum, Plasma or Whole Blood Rapid Test

- In-Vitro diagnostic immune-chromatographic assay of detection and differentiation of HIV Av/Ag 4th Gen antibodies of HIV virus.
- Run with Human Serum, Plasma or Whole Blood
- Qualitative detection of antibodies of HIV-1 (including subtype-o) and HIV-2, as well as p24 antigen of HIV.
- Lateral flow immunoassay with colloidal gold conjugated recombinant proteins for precise results.
- Quick and easy to use, providing accurate results in a short time.
- Detects both antibodies and p24 antigen, ensuring a comprehensive understanding of HIV status.



READING TEST RESULTS

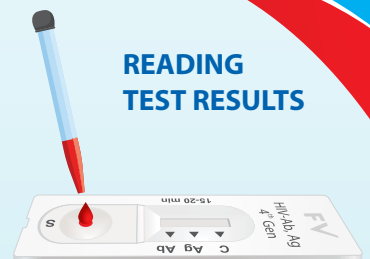


Add 1 drops (10 µL) of serum/ plasma or (10 µL) whole blood in sample well "S" with the micro pipette.




Add 2 drop of the Assay buffer


Negative

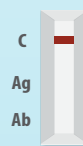

Positive

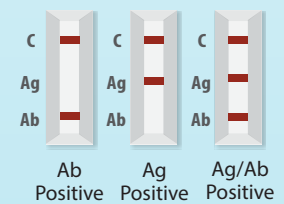

Invalid

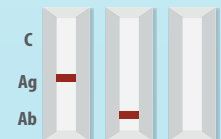

Do not read the results after 30 minutes

ISO 9001 CERTIFIED GMP CERTIFIED ISO 13485 CERTIFIED



PCTSI

Imported and Distributed By :

Phil Capital Trading Solutions Inc.

904 Atlanta Centre, 31 Annapolis St. Greenhills, San Juan City, Metro Manila
T. (02)7001 0770 | E. philtradinginc6@gmail.com