

An Innovative Rapid Test Designed by WANTAI

Rapid Test for Antibody to (HIV 1+2) Human Immunodeficiency Virus

CE 0123



WANTAI

Leading diagnostic test manufacturer of infectious diseases

Your Global Partner in IVD

*Developing Scientifically
Focusing on the Health*



Imported and Distributed By :

Phil Capital Trading Solutions Inc.

904 Atlanta Centre, 31 Annapolis St. Greenhills, San Juan City, Metro Manila
T. (02)7001 0770 | E. philtradinginc6@gmail.com



Innovative



High Performance



Easy-to-use




Rapid

Test Process

1. Prepare Test


Allow the test cassette to reach room temperature, use it within 20 minutes after opening the pouch.

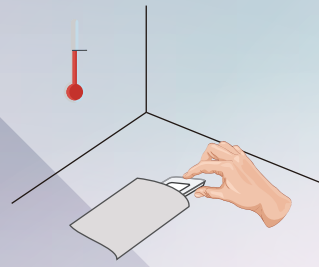


2. Add Specimen

1) For whole blood specimens
Add one drop of blood, then
Add one drop of Diluent Buffer


2) For the serum/ plasma specimens
Add two drops of specimen

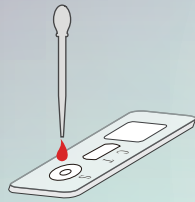

50µl of specimens

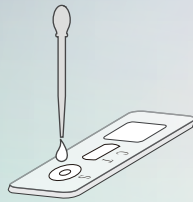


50µl of diluent buffer

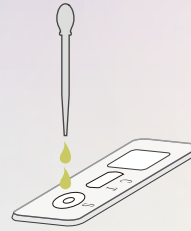

80µl of specimens of S/P

3. Read Result


Read the results within 10-30 minutes.

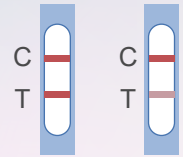

Positive



Negative

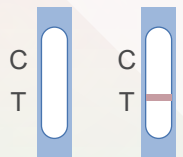

Invalid

* Please refer to product insert for detailed instructions.

Advanced Diagnostic Technology

- 01 **Lateral Flow Technology:** Makes it possible to perform rapid on-site screening
- 02 **Superior Performance:** 100% Sensitivity and 99.54% Specificity. Wantai reacts positive to HIV-1 subtypes A, B, C, D, F, G, H, J, K, O, and HIV-2. The test can detect all HIV subtypes in 1st Int.panel for HIV [NIBSC].
- 03 **Easy-to-use:** No special equipment. Works with whole blood, serum and plasma samples
- 04 **Fast:** Clear answer within 10-30 minutes
- 05 **All in one:** All accessories in the kit allows to do point-of-care testing
- 06 **Room Temperature Storage:** 18 months shelf-life

Order Information

Components	WJ-1810	WJ-1850		
Test Cassette	x 10	x 50		
Diluent Buffer	x 1 vial	x3 vials		