

FirstView®

One Step Rapid Test Device

Bioline
DIAGNOSTICS
Your safety. Our commitment.


Rapid test for Dengue IgG/IgM

**For Qualitative
Detection of
Dengue IgM and IgG
antibodies In human
serum, plasma
or whole blood**


- Test provides an excellent kits for the differential detection of IgG and IgM against Dengue virus Type I, II, III and IV.
- Run with WHOLE BLOOD or PLASMA or SERUM
- Rapid test result in just 15 minutes.
- An immunochromatography assay designed for the qualitative detection and differentiation of specific IgM and IgG antibodies to dengue virus.




READING TEST RESULTS

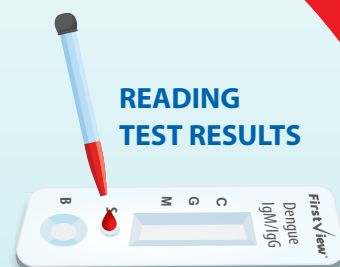

Add 1 drops (5 μ L) of Serum, Plasma, Whole Blood into sample well "S" using the dropper provided.

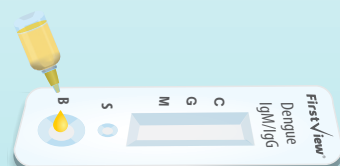

add 2 Drops of buffer (approximately 80 μ L) into the buffer well (B).

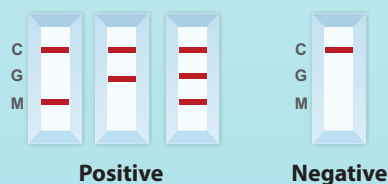

PCTSI

Imported and Distributed By:
Phil Capital Trading Solutions Inc.
904 Atlanta Centre, 31 Annapolis St. Greenhills,
San Juan City, Metro Manila
T. (02)7001 0770 | E. philtradinginc6@gmail.com



IVD