

# Firstview<sup>®</sup>

One Step Rapid Test Device

## Rapid test for Leptospira IgM


**Bioline**  
DIAGNOSTICS  
Your safety. Our commitment.

**For Qualitative  
Detection of  
Leptospira IgM  
Human Serum,  
Plasma  
or Whole Blood  
Rapid Test**

- **Rapid Detection:** Quickly detects Leptospira-specific IgM antibodies in human serum, plasma, or whole blood
- **Timely Diagnosis:** Enables early detection and treatment of Leptospirosis, reducing risk of kidney and liver damage
- **Easy to Use:** Simple immunochromatographic assay for qualitative detection
- **Accurate Results:** Detects IgM antibodies for reliable diagnosis of Leptospirosis
- **Important for Public Health:** Helps control the spread of Leptospirosis through timely diagnosis and treatment



### READING TEST RESULTS

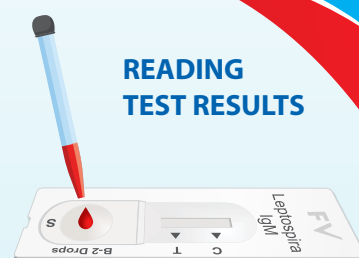

Add 1 drop (10  $\mu$ L) of serum/ plasma or (20  $\mu$ L) whole blood in sample well "S" with the micro pipette.




Add 2 drop of the Assay buffer


#### Negative



#### Positive

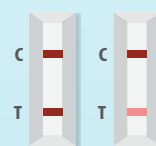

#### Invalid

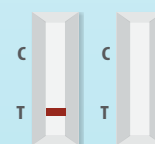

Do not read the results after 20 minutes

ISO 9001  
CERTIFIED

GMP  
CERTIFIED

ISO 13485  
CERTIFIED



**PCTSI**

Imported and Distributed By :  
**Phil Capital Trading Solutions Inc.**

904 Atlanta Centre, 31 Annapolis St. Greenhills,  
San Juan City, Metro Manila  
T. (02)7001 0770 | E. philtradinginc6@gmail.com